

# Quick Reference Guide

# H O R U S   S C O P E

## Step 1

- Slide upward from charging base OR remove from USB charger.



## Step 2

- Ensure the device is adequately charged (solid blue light on power indicator/handle when the device is powered on).



## Step 3

Attach appropriate viewing lens as follows:

- Ensure the device is powered off.
- Find and match the lens alignment marks (small black circles).
- Insert the lens into the Horus Scope connector interface.
- Rotate the lens attachment gently clockwise until it stops.
- Ensure the white position bars are in the appropriate horizontal position.



## Step 4

Connect device to Telehealth unit as follows:

- Connect the AV (out) cord with the converter.
- Connect the converter to the Telehealth unit.
- Some units may require an additional cable to connect the Telehealth unit to the converter (e.g., SX20).



### Step 5

- Press the power button to begin using the camera.



### Step 6

- Press “OK” button to freeze an image.



### Step 7

Adjust brightness/focus as follows:

- Brightness controls on the front (under “OK” button) adjust amount of LED light emitted from the device.
- Brightness controls also act as zoom in/out buttons for still images.
- Focus wheel (found on back) adjusts image quality.

### Step 8

- Power off the device.

### Step 9

- Disinfect the device with an alcohol wipe.
- Do not sterilize the device.
- Clean the lens with dust free cleaning cloth.



### Step 10

- Recharge device (power indicator light will turn off indicating device is fully charged while connected to charger).
- Flashing blue light (on power indicator/handle) means battery is less than 25% full when unit is on.
- Orange light means battery is less than 100% full when connected to charger.



## Tips

- Horus Scope will not display an image if there is no lens attached.
- Do not touch the lens.
- Images/videos are not stored/recorded during Telehealth appointments.
- Remove battery if device will be stored for more than two weeks.