


Once an exam is selected and it opens to view in ZFP, note the following:

1. The report palette does not launch with the image. To display the report, you must click the report icon.
2. The exam that first launches is labelled as **PRIMARY** in the top right corner of the image.
3. If a comparative exam is available, it will be listed in the left side panel above the series for the current exam. To search comparative exams of the same modality, click the Modality Filter icon.




To compare exams for the patient:

1. Select the comparative exam to bring the series in view for that exam.
2. Click and drag the desired image into the viewport.

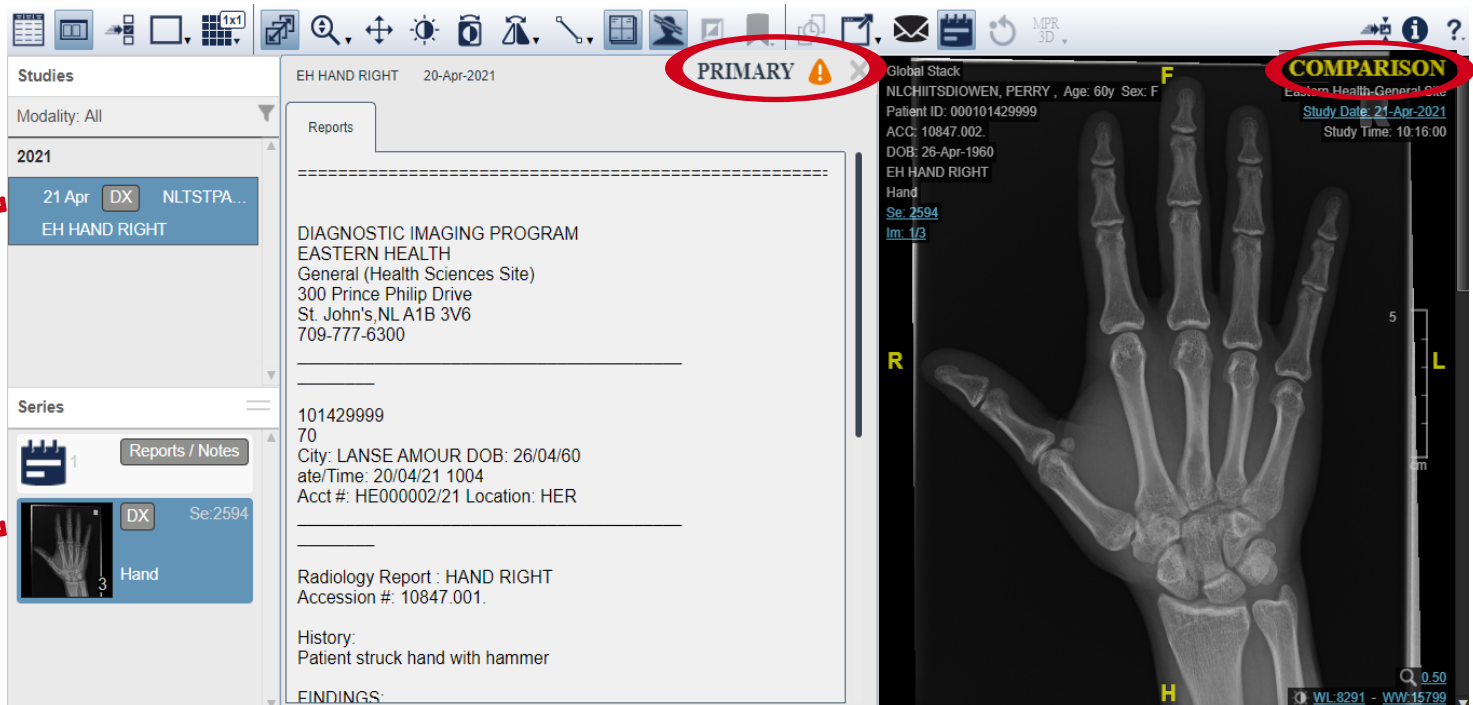

**PLEASE NOTE:** If opened, the report palette **does not** automatically sync with the comparative image view. To view the report for the comparative exam, you must click on the report icon **each time**. Look for the **COMPARISON** indicator at the top of images/reports as a reminder that you are no longer looking at the original exam opened.

**NOTE:** The warning icon  indicates that all images and reports may not be matching.